


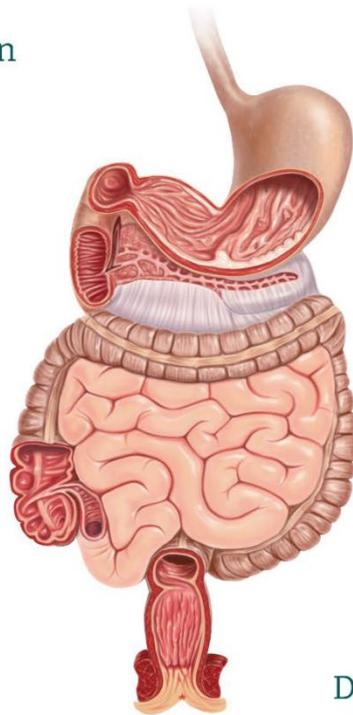

## EverLift® Submucosal Lifting Agent

### Procedure for preparation of EverLift


- Remove the cap from the tube and remove the syringe.
- Examine the syringe to verify there is no damage.
- Using aseptic technique, attach the syringe to the luer fitting on the endoscopic injection needle (not provided).
- Prime the needle with EverLift® prior to injection.
- With the needle retracted, insert the needle's catheter through the working channel of the scope.



### Proper Technique for Injecting EverLift

- Insert the tip of the injection needle at a 30°- 45° angle to the surface into the submucosal space of the gastrointestinal tract, ensuring the tip of the needle is entirely beneath the mucosa.
- The maximum allowable dose is 50 mL per patient.
- Do not insert the needle perpendicular to the colon surface, as this may lead to transmural injection of the lifting agent directly into the peritoneal cavity.

### The Benefits of EverLift

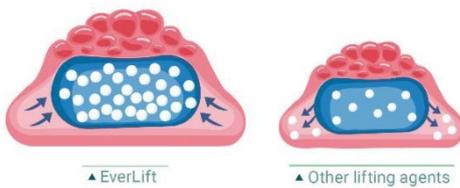

### Does EverLift produce artifacts in pathology post EMR/ESD?

Laborie's 2021 post-market clinical follow-up study of its 5 mL and 10 mL syringes validate that EverLift Submucosal Lifting Agent does not produce artifacts.

Read the whitepaper by scanning the QR code.



Order No.	Description	Quantity
BGIS-55S	5mL Syringe - EverLift Submucosal Lifting Agent	Each
BGIS-55	5mL Syringe - EverLift Submucosal Lifting Agent	Bx10
BGIS-59S	10mL Syringe - EverLift Submucosal Lifting Agent	Each
BGIS-59	10mL Syringe - EverLift Submucosal Lifting Agent	Bx10

**ENDOMED PTY. LTD.**  
SPECIALISING IN ENDOSCOPY

A.B.N. 98 050 413 379

[orders@endomed.com.au](mailto:orders@endomed.com.au)

ENDOMED PTY. LTD.  
POST OFFICE BOX 445,  
STRATHPINE QLD. 4500

TELEPHONE 07 3881 1883

1800 773 474

FACSIMILE 07 3881 0211