

# Spot<sup>®</sup> Ex Endoscopic Tattoo


Take The Next Step in the Fight Against Colon Cancer



## Guide to Tattoo Placement<sup>3</sup>


### For Surgical Resection

- Place the tattoos in 2-3 quadrants, circumferentially;
- 3-5 cm from the lesion on the distal (anal) side

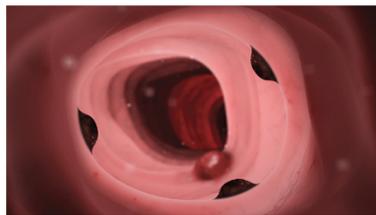


### For Future Endoscopic Resection

- Place a single tattoo;
- 3-5 cm from the lesion on the distal (anal) side

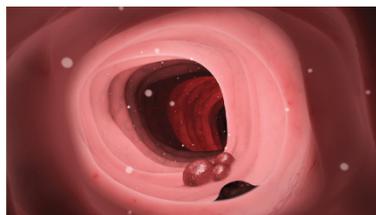


### For Surveillance of Large/ Piecemeal Resected Lesions

- Place a single tattoo after resection of the lesion;
- 3-5 cm from the lesion on the distal (anal) side

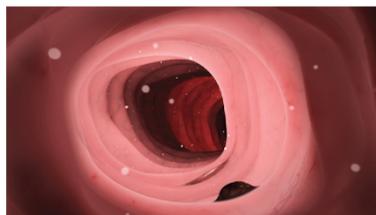

ESGE recommends that tattoos be placed  $\geq 3-5$ cm anatomically distal (anal side) to the lesion. In order to locate polyps or scars at a subsequent endoscopy, in general only one tattoo is needed.<sup>1</sup>

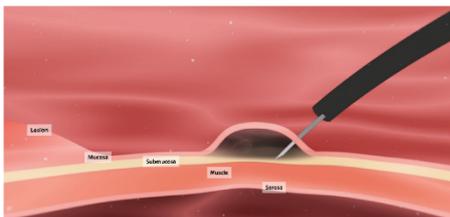

1. Place injection 3-5 cm distal (anal side) of the area of lesion.
2. Inject tangentially, at a 30-45° angle to the mucosa.
3. Create a saline bleb to find the submucosal plane prior to injecting Spot Ex to reduce risk of transmural injection.
4. Use 0.5-0.75 mL of Spot Ex per injection site, and no more than 8 mL per patient.<sup>4</sup>

1. Ferlitsch et al. Colorectal polypectomy and endoscopic mucosal resection: European Society of Gastrointestinal Endoscopy (ESGE) Guideline – Update 2024. Endoscopy 2024 Jul;56(7):516-545

2. Ferlitsch et al. Colorectal polypectomy and endoscopic mucosal resection (EMR) European Society of Gastrointestinal Endoscopy (ESGE) Clinical Guideline. Endoscopy 2017; 49; 270-297

3. Medina-prado, et al. When and how to use endoscopic tattoo in the colon Clinical Gastroenterology and Hepatology 2021;19:1038–1050

4. Spot Ex IFU G46-006 (LBL01906 [K])

Spot is a registered trademark of GI Supply, Inc. © 2024 Laborie Medical Technologies Corp. All Rights Reserved.

Order No.	Description	Quantity
BSPOTEX	Endoscopic Tattoo - 5mL Syringe	Bx10

**ENDOMED PTY. LTD.**  
SPECIALISING IN ENDOSCOPY

A.B.N. 98 050 413 379

[orders@endomed.com.au](mailto:orders@endomed.com.au)

ENDOMED PTY. LTD.  
POST OFFICE BOX 445,  
STRATHPINE QLD. 4500

TELEPHONE 07 3881 1883

1800 773 474

FACSIMILE 07 3881 0211